

CARSON®

MJ-50

MagniFold™

2x LED Lighted Magnifier

Congratulations on selecting your new MagniFold! In order to achieve optimum performance, please follow instructions for proper use and care.



PRODUCT SPECIFICATION

Magnification:	2x	Dimensions:	5.7" x 0.8" x 7.7"
Weight:	0.32 lbs	Battery:	3 G10 Button Cell (included)


USING MAGNIFIER

1. Move handle to open position and place magnifier over the subject you are viewing.


For Best Results:

- Depending on your vision, you may need to move the magnifier closer or further away until focused.

2. When finished viewing, return handle to closed position.

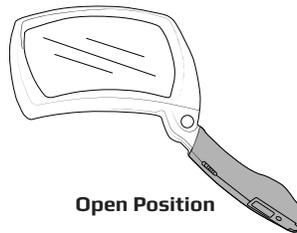


Open Position

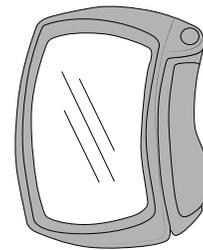

Closed Position


TURNING ON LED

Slide switch to turn LED light ON/OFF.

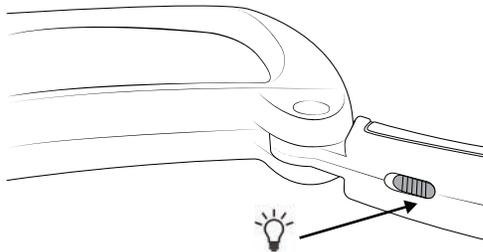

REPLACING BATTERIES

Loosen screw on battery door with a phillips #0 screwdriver and remove door. Insert 3 new G10 batteries in the correct polarity as indicated inside battery compartment. Replace battery door and tighten with screwdriver.

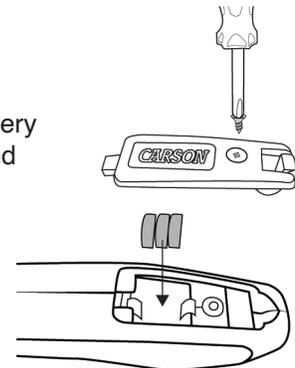

WARNING

Batteries: Mixing old and new, different brands or types within brands may cause batteries to leak, overheat or explode. Keep away from fire or excessive heat. Inspect batteries for corrosion or leakage. Insert in direction indicated. Discard button cell batteries carefully. Keep button cell batteries away from children.

CARE

Clean using microfiber lens cloth. Do not use any abrasives, chemical cleaners, or solvents.

Customer Service: If you experience any difficulties, please contact us and we will be happy to help you. For warranty information, visit www.carson.com/warranty



✉ info@carson.com



✉ uksupport@carson.com



✉ eusupport@carson.com

www.carson.com

Carson Optical 2070 5th Avenue,
Ronkonkoma, NY 11779
1-631-963-5000