

### 2.5x Clip-On Medicine Bottle Magnifier

Congratulations on selecting your new MagRx! In order to achieve optimum performance, please follow instructions for proper use and care.





### PRODUCT SPECIFICATION

|                     |                  |                                |
|---------------------|------------------|--------------------------------|
| Magnification: 2.5x | Weight: 0.06 lbs | Dimensions: 2.3" x 2.5" x 1.5" |
|---------------------|------------------|--------------------------------|

### USING MAGNIFIER

1. Move magnifier to open position. Lift arms to insert medicine bottle. Medicine bottle will rest against holder.
2. Adjust magnifier over label. Depending on your vision, you may need to move the magnifier closer or further away until focused.

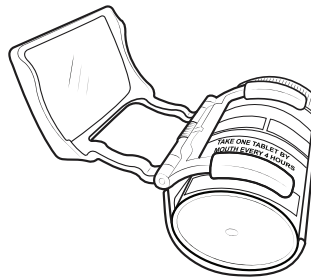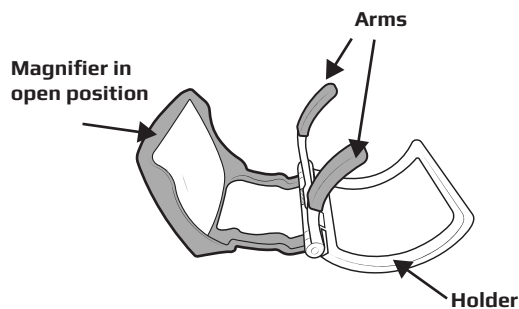

### CARE

Clean using microfiber lens cloth. Do not use any abrasives, chemical cleaners, or solvents.

#### Customer Service:

If you experience any difficulties, please contact us and we will be happy to help you.



✉ [info@carson.com](mailto:info@carson.com)



✉ [uksupport@carson.com](mailto:uksupport@carson.com)



✉ [eusupport@carson.com](mailto:eusupport@carson.com)

For warranty information, visit [www.carson.com/warranty](http://www.carson.com/warranty)

**[www.carson.com](http://www.carson.com)**

Carson Optical 2070 5th Avenue, Ronkonkoma, NY 11779 | Tel: 631-963-5000